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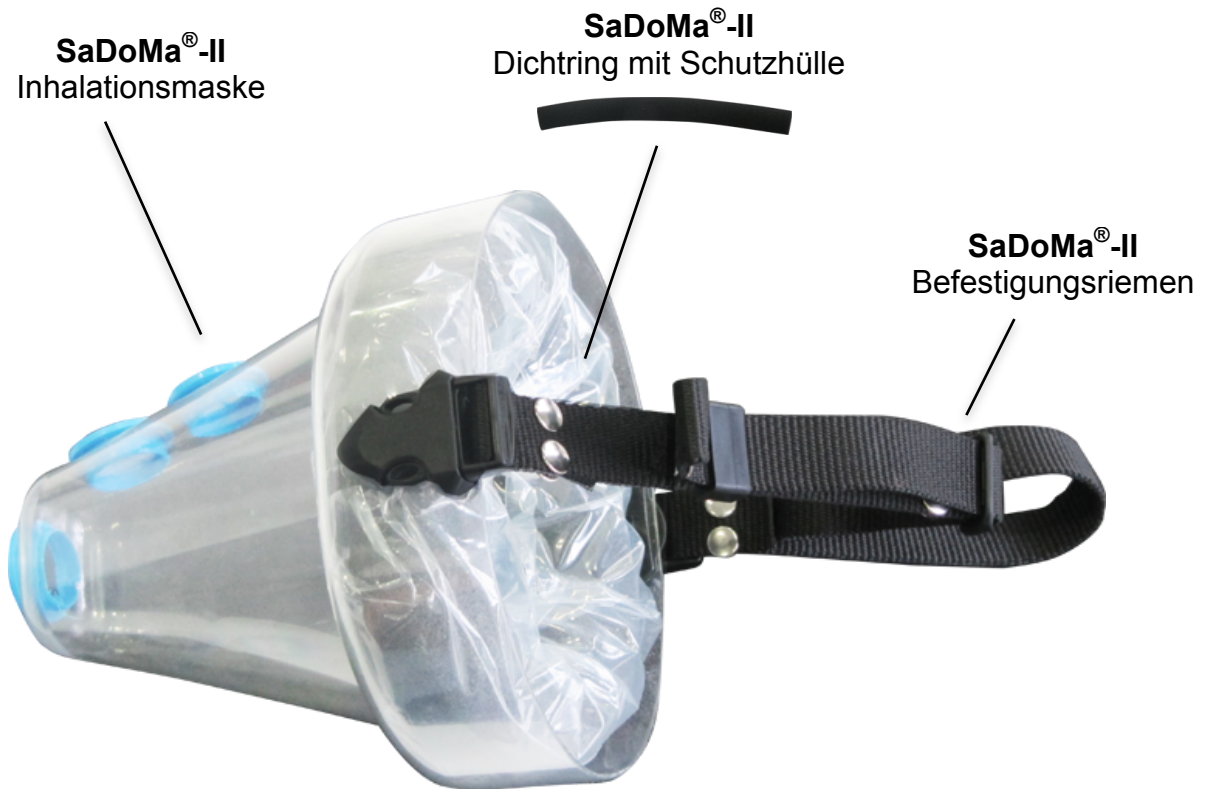
Operating Manual

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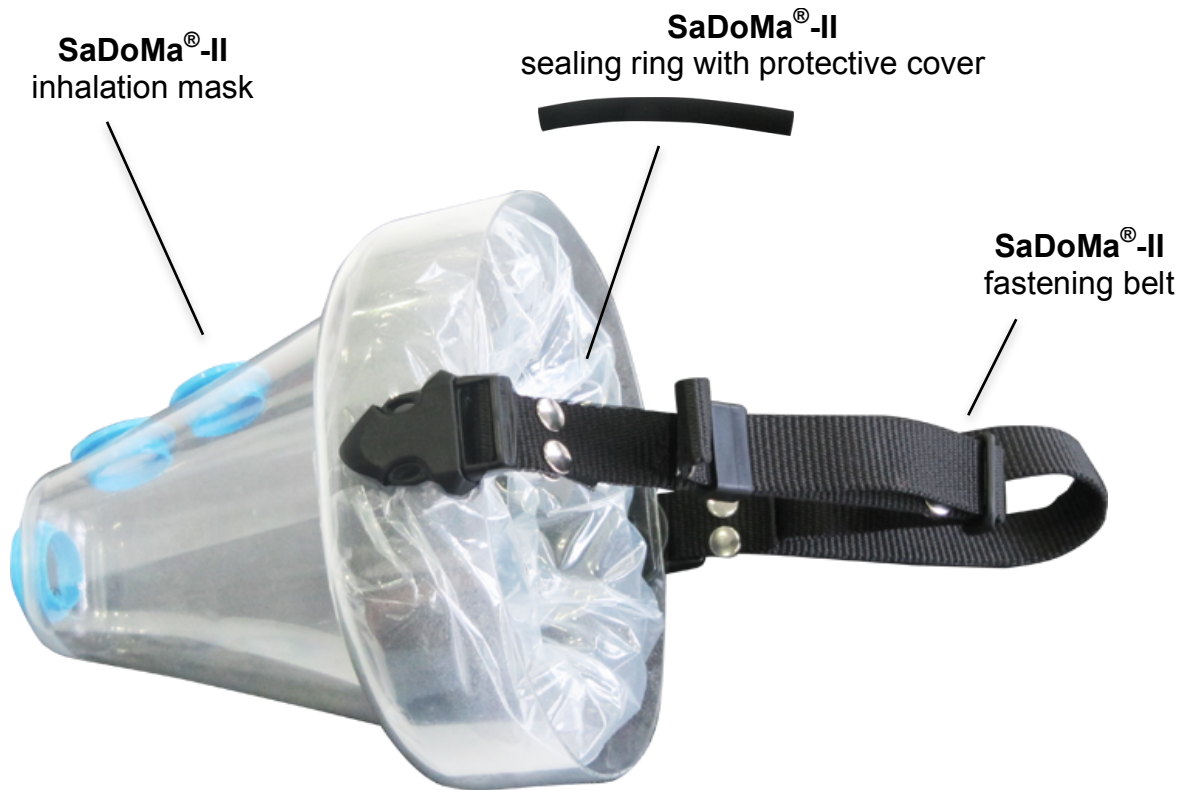
### How to operate the SaDoMa®-II inhalation mask

1. Removal of the SaDoMa®-II inhalation mask from its packaging  
Remove the SaDoMa®-II inhalation mask from its packaging together with the separately packed fastening belt inside. Unpack the fastening belt and connect it to the SaDoMa®-II inhalation mask.
2. How to attach the SaDoMa®-II inhalation mask  
Now put the SaDoMa®-II inhalation mask over the dog's snout and attach the SaDoMa®-II inhalation mask to the dog's head by means of the fastening belt.

3. Connection of the SaDoMa®-II inhalation mask with the M-neb® vet  
If you are using the SaDoMa®-II inhalation mask together with the M-neb® vet, please follow the operating manual of the M-neb® vet.

#### **Cleaning the SaDoMa®-II inhalation mask**

4. Cleaning the SaDoMa®-II inhalation mask  
The SaDoMa®-II inhalation mask should be cleaned after completion of every single inhalation by means of lukewarm water. For this purpose, please remove the fastening belt and sealing ring with protective cover. The M-neb® vet nebulization unit also needs to be removed from the inhalation mask before starting the cleaning process.
5. Replacement of the protective cover  
The protective cover serves as a protection of the sealing ring. For hygienic reasons, the protective cover should be replaced after approx. 5 inhalations depending on the degree of contamination.

Operating Manual of SaDoMa®-II

Article-No.:	Image:	Description:
DM-200		<p><b>SaDoMa®-II</b> set consisting of inhalation mask, sealing ring with protective cover and fastening belt</p>
DM-201		<p><b>SaDoMa®-II</b> protective cover for sealing ring</p>
DM-202		<p><b>SaDoMa®-II</b> fastening belt</p>
DM-203		<p><b>SaDoMa®-II</b> sealing ring</p>

**Nebu-Tec International**  
**med. Produkte Eike Kern GmbH**  
Kreuzfeldring 17  
63820 Elsenfeld - GERMANY

Tel.: +49 (0) 6022 – 610-62-0  
Fax: +49 (0) 6022 – 610-62-99

Email: [info@nebu-tec.de](mailto:info@nebu-tec.de)  
Web: [www.nebu-tec.de](http://www.nebu-tec.de)