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Please note:

**Our Quick Start Guide does not replace reading and observation of the detailed operating manual.**

You will find there more details as well as information about changing the program, cleaning and how to charge the ultrasonic nebulizer.

1

1a. Put the sealing ring and the inhalation mask over the horses muzzle and secure the fastening strap by plugging the two quick-release fasteners into the counterpart.

2

2a. Slide the ultrasonic nebulizer along the slide rails until it snaps in.

2b. Insert a filter membrane and slide the filter basket over the two bolts of the mask.

3

3a. Fill the prescribed substance into the mesh nebulization unit.  
**Please read the detailed operating manual!**

4

4a. Connect the mesh nebulization unit by leading the connection cable to one of the two sockets of the ultrasonic nebulizer.

4b. Slide the protection cover along the side rails until it clicks into place.

5

5a. Turn on the ultrasonic nebulizer by holding down the power button through the opening in the protection cover.

5b. The ultrasonic nebulizer switches off automatically as soon as there is no longer any residual quantity.